TISSUE REGENERATION

REXON  **AGE 2**

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rexon  **AGE2**

WHERE NO ONE HAS GONE BEFORE

LA RISONANZA QUANTICA MOLECOLARE

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QUANTUM MOLECULAR RESONANCE

Tissue Regeneration

QUANTUM MOLECULAR RESONANCE APPLIED TO AESTHETIC MEDICINE

REXON-AGE 2 is newly designed Medical Device that finds application in aesthetic medicine and whose operating principle is based on the innovative technology patented by Telea Medical as QMR (Quantum Molecular Resonance). Unlike the more traditional radio frequency devices, the operating principle of the REXON-AGE 2 is NOT BASED ON THE EFFECT of HYPERTHERMIA (heating).

REXON-AGE 2 generates 16 different frequencies transmitted simultaneously to the patient. The working principle of REXON-AGE 2 is intimately related to the ability with which these different frequencies STIMULATE, through a synergistic action, the CELL METABOLISM.

To understand the importance of using multiple frequencies, think to each of them as if it were a “key” activating specific cell receptors and stimulating their activity. It has been well understood that the overall action of a large number of frequencies, which act simultaneously on the multiple cellular receptors, is able to induce a very effective and deep stimulation.

Additionally, the uniqueness of the QMR technology employed by REXON-AGE 2 is linked to the fact that the frequencies used are much higher than those generally used by common radio frequency devices. The latter, in fact, typically uses a single frequency around 0.5 MHz (with the sole purpose to induce hyperthermia effect); REXON-AGE 2 instead generates a spectrum of more than 16 frequencies between 4 MHz and 64 MHz, that can therefore stimulate the natural cellular activity in a much more sophisticated way, effectively and less invasively.

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*It is a **pleasant therapy** for the patient, **non-invasive**, which offers ample space for customization and **great flexibility** in the various modes of use. The standard protocol provides a general indication concerning which parameters should be used with each of the several electrodes available and with respect to the area of the body or face to be treated. It is therefore left to the discretion of the operator, **case by case**, the choice of the most appropriate parameters.*

*In defining the setting of the most suitable parameters of the therapy, it must be held in the highest consideration the **sensation perceived** by the patient. In order to optimize the treatment it is important that the patient perceives a **pleasant and comfortable warmth**; it never has to be unpleasant nor annoying.*

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Underlying once again that the therapy with REXON-AGE 2 permits a great degree of freedom and customization, depending from the Patient's needs, we report here below a standard approach:

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Round Electrodes

Round Electrodes with diameter of 7 cm or of 5 cm

Eye Contour electrodes, dimension 7 x 2.4 cm

Once the electrodes have been put in position and the parameters have been set on the unit, the therapy can proceed without any further intervention from the operator.

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Q-Touch Gloves

Constitute an extremely valuable and interesting accessory because they allow the operators to transfer the therapy to the patient directly from their hands. With Q-Touch gloves, the therapy is both very effective and extremely pleasant for the patient.

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Handpiece

It is also available the more classical method of use, that is, through the Handpiece Electrode. Two different sizes of the tip are available, 35 mm and 50 mm respectively.

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Creams

*Non-conductive **neutral Cream**, which allows the Handpiece Electrodes and the Q-Touch Gloves to gently glide on the patient's skin without any friction, for the comfort of the Patient. **Moisturizing Cream** with low molecular weight and fast absorbing properties, which allows a deep hydration of the skin, ideal to be used immediately after the session with REXON-AGE 2.*

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AIM OF THE THERAPY AND SCOPE OF APPLICATION

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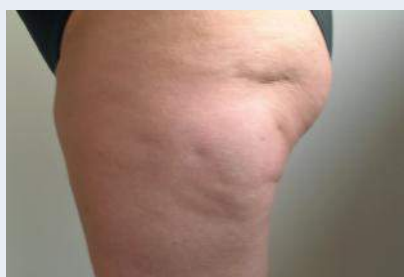
- * Increase of the content of collagen and elastin in the dermis*
- * Skin rejuvenation effect and the dermis (commonly called "natural facelift")*
- * Firming and increase skin elasticity*
- * Elimination or significant wrinkle reduction*
- * Important improvement of the skin texture*

AND ALSO

- * Significant reduction of active acne*
- * Reduction of cellulite, also due to the improvement of the lymphatic microcirculation*
- * Toning up the skin laxity in areas such as abdomen, inner thighs, upper arms, buttocks*

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*The stimulation with the use of **QMR** technology induces interesting metabolic responses*

- *Involvement of membrane receptors*
- *Increased production of collagen*
- *Increased production of elastin*
- *Increase of the derma thickness up to a 20%*

It follows that:

*The medical aesthetic therapy with **REXON-AGE 2** is designed to stimulate the natural cell metabolism, making sure that even a skin no longer young recovers **vigor and tone**.*

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Reached the age around 25 years, the production of important structural proteins in the dermis (collagen and elastin) decreases and the protein content tends to decrease due to the slowing down of the metabolic activity of the cells (as the fibroblasts).

*The medical therapy with **REXON-AGE 2** is designed to thwart this aging process, leading back the tone of the **dermis to a renewed vitality**.*

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In order to ensure maximum safety for the patient, Telea Medical, in collaboration with Research Institutes and National Universities, has conducted a series of trials and studies in order to make sure with certainty that the stimulation through QMR technology is totally safe and does not induce any kind of cellular damage.

WE DO NOT JUST PROVIDE GENERIC INFORMATION FROM THE INTERNET TO STATE THAT OUR THERAPY WORKS.

Researches and numerous studies have been conducted on QMR therapy, in order to prove its validity.

FOR THE SAFETY OF YOUR PATIENTS, WE INVITE YOU TO PROPERLY CHECK THE INFORMATION PROVIDED BY THE OTHER MANUFACTURERS.

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CELLULAR STIMULATION INDUCED BY QMR

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*A single muscle cell stimulated by the **QMR** undergoes a series of contractions and relaxations, depending on whether the field is turned on or off.*

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*By using a fluorescence probe was observed that when the **QMR** is activated the membrane potential is depolarizing and then returns to its "resting" potential when the **QMR** is interrupted.*

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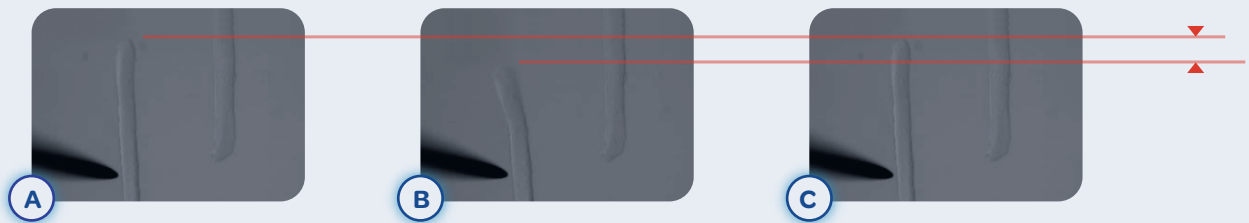
*By using fluorescence probe of the muscle cells stimulated by the **QMR** was observed a release of calcium ions from the sarcoplasmic reticulum, which means that intracellular pathways calcium-dependent has been activated.*

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
*The following pictures A,B, C show a single muscle cell being stimulated by the **QMR**. The cell misshapes when the **QMR** electrical field emitted from the tip of a tiny electrode is on (Picture B) and then returns to its original shape when the stimulation is interrupted (Picture C). This stimulation, which is a sort of a "cellular massage", has an impact on the different membrane proteins (receptors), intended to detect mechanical type of stress and induce a cell mitosis process.*

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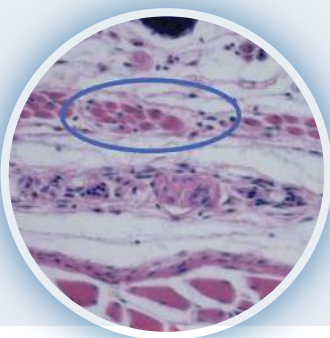*Finally, one of the university studies on the tissue showed that after a course of a **QMR** stimulation the parvocellular infiltrate (black dots) were observed inside the muscle bundles.*

This is a result of the regenerative process in progress.

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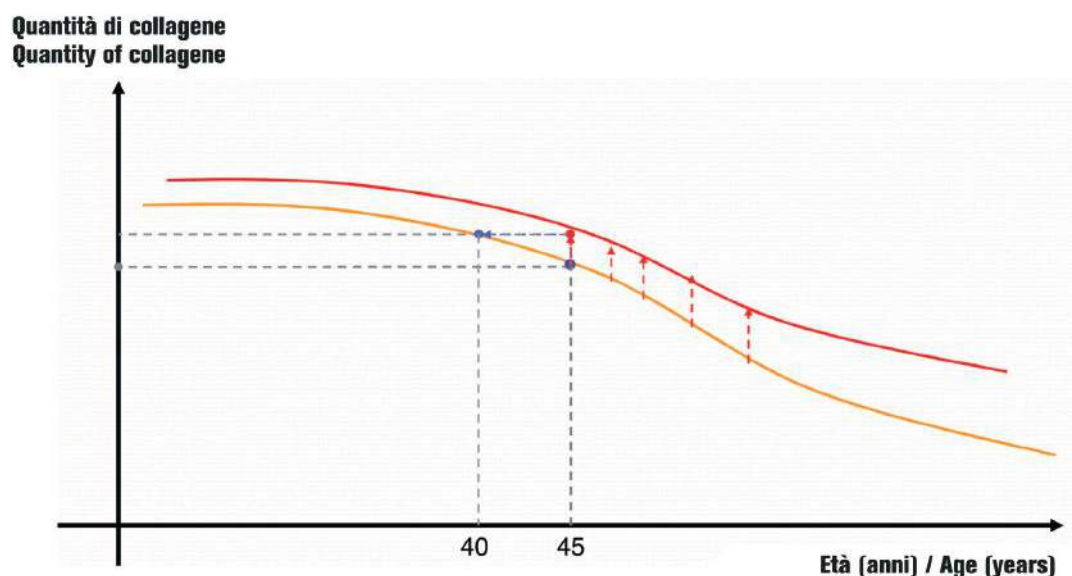
HOW TO SLOW DOWN THE AGING PROCESS OF THE SKIN

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
When reached a certain age, usually around 25 years old, the production activities of major structural proteins contained in the dermis (mainly collagen and elastin) begin to slow down. These proteins act as a "scaffold" and, as such, they are essential to support the structures of the skin. Their abundance in young skin determines its typical tone, texture and elasticity. As we age, the protein content tends to decline constantly, due to the slowdown of cell metabolism, particularly fibroblasts. Treatment with REXON-AGE 2 aims to accelerate the cell metabolism, making sure that even a skin which is no more young behaves again as if it was still so.

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
The graph describes a qualitative indication on how it varies with age the proteins content in the dermis, mainly collagen and elastin (orange curve), and how this content is modified by the therapy with REXON-AGE 2, thus slowing down the aging process (red curve).

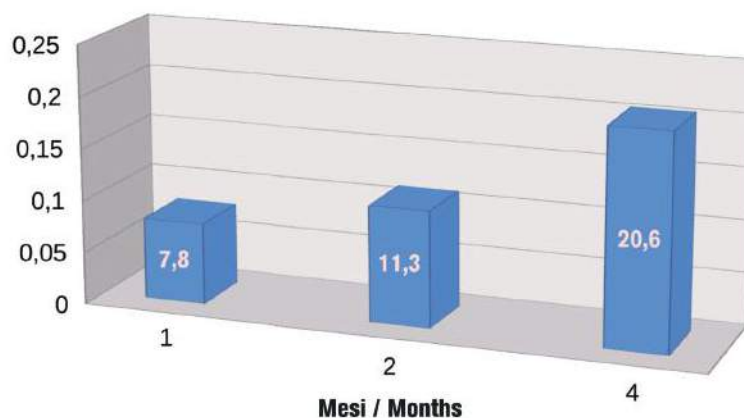
Media variazione assoluta (%) - mandibola Thickness mean variation (%) - jaw area

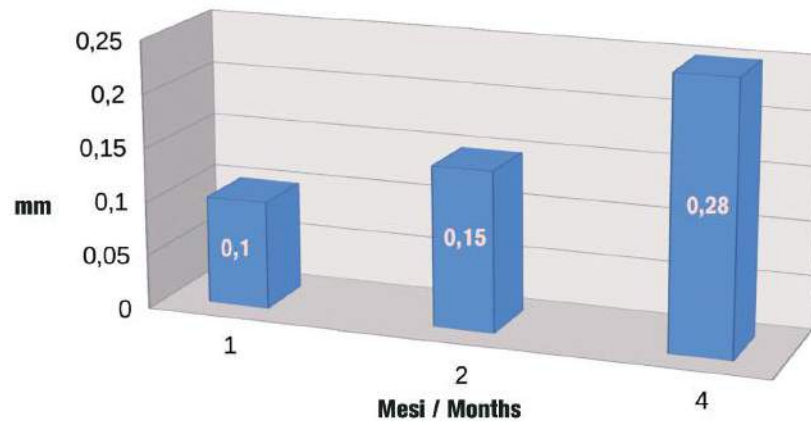
Media variazione assoluta (mm) - mandibola
Thickness mean variation (mm) - jaw area

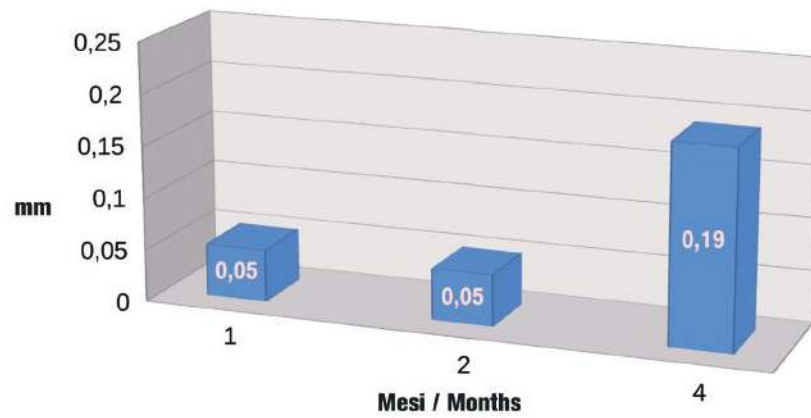Media variazione assoluta (mm) - zigomo
Thickness mean variation (mm) - zygomatic area

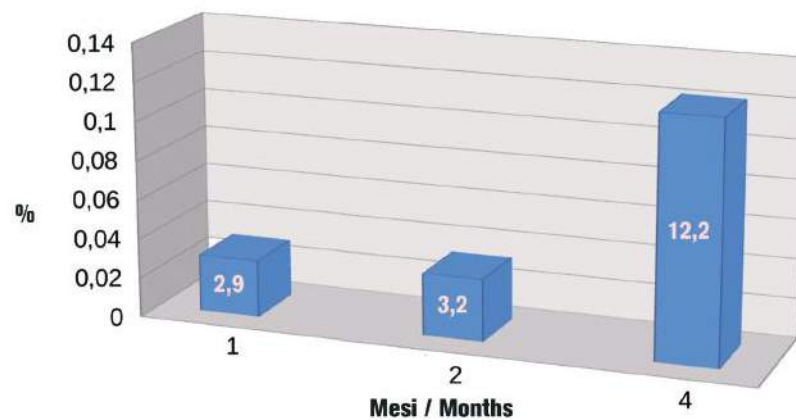Media variazione assoluta (%) - zigomo
Thickness mean variation (%) - zygomatic area

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SCIENTIFIC STUDIES AND COOPERATION

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Telea Electronic Engineering S.r.l.

Via Leonardo Da Vinci, 13
36066 Sandrigo - VI - ITALY
Tel. +39 0444 239519

info@teleamedical.com
www.teleamedical.com