Telea

MEDICAL

VESALIUS MCN PLUS

THE QUANTUM MOLECULAR RESONANCE GENERATOR

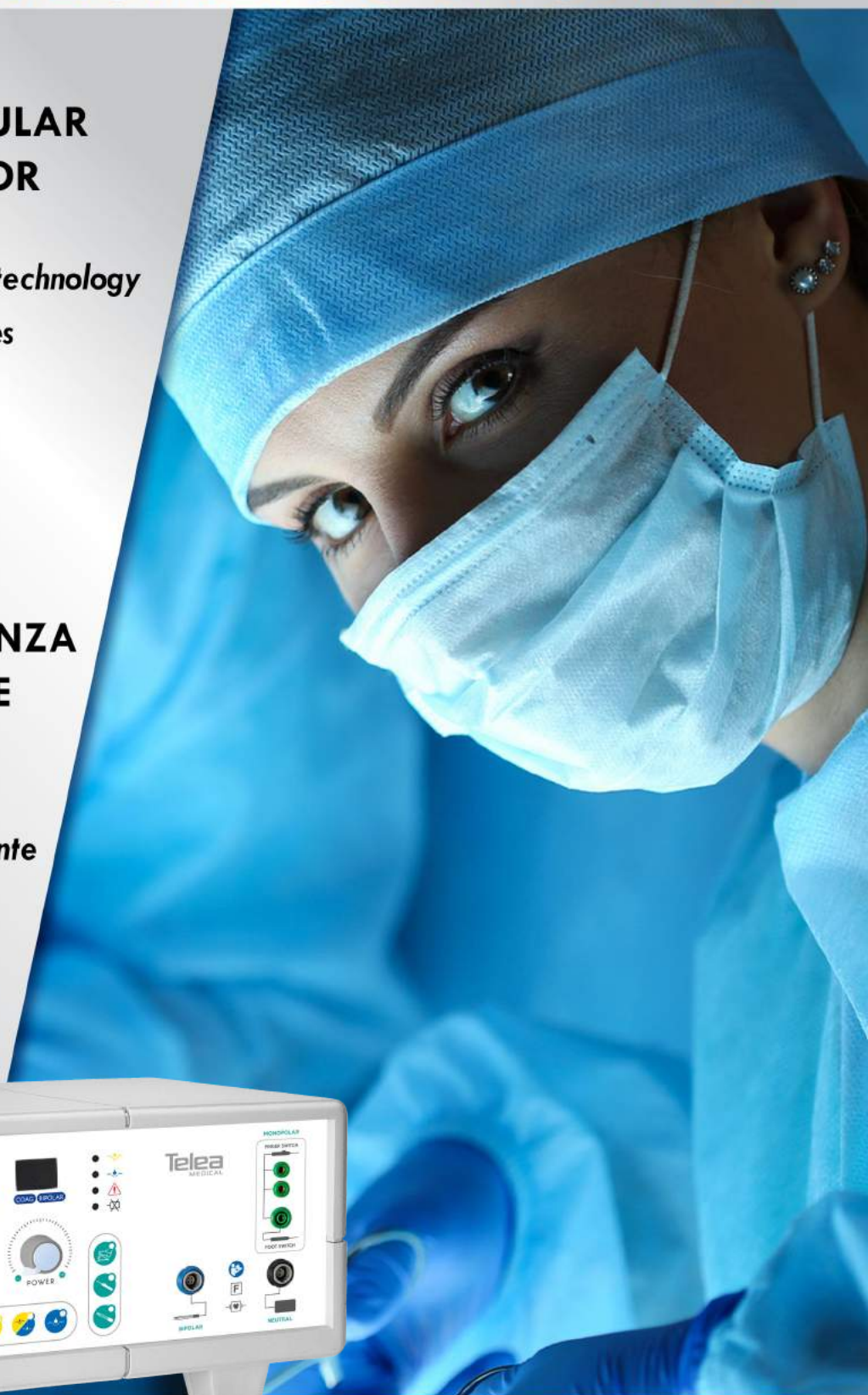
QMR: an innovative and unique technology for the human health that improves the quality of life

RESEARCH AND DEVELOPMENT
FOR THE PATIENT

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VESALIUS MCN PLUS

Product code/codice: 2501023

THE QUANTUM MOLECULAR RESONANCE TECHNOLOGY

QMR technology: QMR is a unique and patented technology: a special combination of different frequencies (Quantized High Frequency Spectrum) that allows the cut and the coagulation thanks to a resonance effect on the tissues. The cutting function causes a breakdown of molecular bonds at low temperature; this feature allows a cold and precise cut and, thanks to a limited thermal damage, permits an extremely delicate incision. Differently from the standard electrosurgical HF equipment, which cause the vessels collapse and tissue necrosis, Vesalius deploys a peculiar waveform that is slightly out of resonance but enough to increase the temperature up to 63-65 °C and to trigger the denaturation processes of the fibrinogen which turns into fibrin and coagulating the blood, without thermal damaging the vessels.

Vesalius MCN plus is a modern mono and bipolar device, purposefully developed for a wide type of operations. It finds its best application in procedures of Neurosurgery, Skull base and Spine Surgery. Vesalius MCNplus includes a wide range of accessories: non-stick high quality, slim and sturdy forceps designed to be used with QMR. The forceps are slender and precise providing excellent control for smaller keyhole access with a variety of tip sizes available.

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BENEFITS*



- Monopolar and Bipolar outputs
- Six different operating functions
- Quantized High Frequency Spectrum: 4-8-12-16MHz
- No necrotic tissue and thermal damage
- Decrease intra-operatively blood loss
- Precise removal of tissue in a practically bloodless field
- No irrigation required

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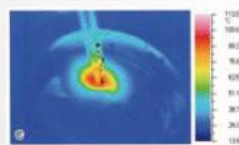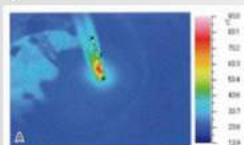Vesalius Non-Stick bipolar forceps

Traditional bipolar forceps

* see evidences on the reference list available at www.teleamedical.com/en/scientific-doc-surgery/

Vesalius is a registered Trademark by Telea Electronic Engineering Srl

Via Leonardo Da Vinci, 13 - 36066 Sandrigo (Vicenza) ITALY - info@teleamedical.com
tel. +39 0444 239519 | fax +39 0444 914319

TELEAMEDICAL.COM